


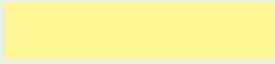








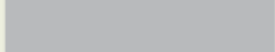
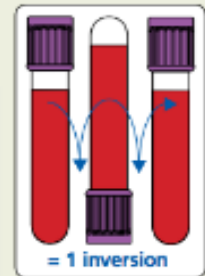


# PHLEBOTOMY ORDER OF DRAW

\* When using a winged blood collection set for venipuncture and a coagulation (citrate) tube is the first specimen tube to be drawn, a discard tube should be drawn first. The discard tube must be used to fill the blood collection set tubing's "dead space" with blood but the discard tube does not need to be completely filled. This important step will ensure proper blood-to-additive ratio. The discard tube should be a nonadditive or coagulation tube.

Closure Color	Collection Tube	Mix by Inverting
<b>BD Vacutainer® Blood Collection Tubes</b> ( <i>glass or plastic</i> )		
	• Blood Cultures - SPS	8 to 10 times
	• Citrate Tube*	3 to 4 times
 or 	• BD Vacutainer® SST™ Gel Separator Tube	5 times
	• Serum Tube ( <i>glass or plastic</i> )	5 times (plastic) none (glass)
	• BD Vacutainer® Rapid Serum Tube (RST)	5 to 6 times
 or 	• BD Vacutainer® PST™ Gel Separator Tube With Heparin	8 to 10 times
	• Heparin Tube	8 to 10 times
 or 	• EDTA Tube	8 to 10 times
	• BD Vacutainer® PPT™ Separator Tube K <sub>2</sub> EDTA with Gel	8 to 10 times
	• Fluoride (glucose) Tube	8 to 10 times

Note: Always follow your facility's protocol for order of draw



Remixed by ResourceNurse.org